

Test Update



Complement Total (CH50) Acceptable Specimen Change, Effective 11/3/15

The specimen requirement for Complement Total (CH50) has changed from serum with or without gel to serum no gel (red top) tube **only**.

Collection instructions:

1. Collect specimen into one serum no gel (red top) tube.
2. Place tube on wet ice immediately after drawing.
3. Spin down and separate from clot
4. Immediately freeze serum.

If you have questions, please contact Dr. Kristin Baer, Chemistry Medical Director (218)249-5208 or Jennifer Viergutz, Laboratory Operations Manager (218-249-5724)